

# Tred-lite Care Information

We trust you are satisfied with your new Tred-lite footwear and they prove to be effective.

The Tred-lite collection by Steeper is available in a variety of fashionable styles, providing comfortable and lightweight footwear that meets your specific needs.

Diabetic specifications

Lightweight, practical and  
customisable footwear

Adaptations on Tred-lite  
and patients' own footwear



Specialist sole  
alternatives

Bespoke insoles ensure  
best clinical outcome

Shock absorbing,  
lightweight insole  
support

# Tred-*lite* Care & Usage Instructions

The Tred-*lite* footwear range is intended for the single exclusive use by the named patient.

The following instructions will help you care for your new Tred-*lite* footwear;

1. Ensure footwear is fastened firmly.
2. Wear for short periods initially.
3. Clean leather with shoe cream or good quality polish regularly. Suede shoes will need a suede brush.
4. Never dry wet shoes using direct heat.
5. Check soles and heels for repair being needed.
6. If you experience any problems with your footwear, contact the hospital department where they were issued.
7. Insoles provided with your footwear should be reviewed/ replaced regularly - your orthotist will advise you.
8. You will be advised of any review processes by your orthotist.

Ref No: .....

Orthotist: .....

Patient Name: .....

Steeper Group,  
Unit 3 Stourton Link  
Intermezzo Drive  
Leeds  
LS10 1DF

Tel: +44 (0) 113 207 0435  
Email: [orthoticenquiries@steepergroup.com](mailto:orthoticenquiries@steepergroup.com)

[www.steepergroup.com](http://www.steepergroup.com)

STPOR019 Issue 2

