

Freedom Highlander®



Engineered for active amputees who require a lower build height foot, the Highlander foot provides a smooth gait and excellent energy return. The split-keel provides stability on uneven terrain by enhancing inversion and eversion. Many transfemoral amputees prefer this foot because of its ideal balance of stability and dynamic response.



***Water-resistant:** Safe for contact with water and occasional submersion. Rinse immediately with fresh water and dry following exposure to water, particularly salt or chlorinated.

Features and Benefits

- Low build height with integral male pyramid
- Split keel design for excellent inversion/eversion
- Weight limit 166kg
- Available in 9 stiffness categories

Cosmetics



Mid profile foot shell

Sandal toe: sizes 22-28cm only



Ordering Information

Trial Period - This foot has a 30 day trial period on sizes 23-30 and categories 2-8 only.

Impact Level Descriptions

Low: Daily activities include mostly level ground walking, moving around in the home and the community.

Moderate: Daily activities include up to unlimited walking, climbing stairs and occasional moderate lifting (this does not include running).

High: Daily activities include fast walking, jogging, running, lifting heavy objects and/or recreational sports.

Ordering Example - NFS3526L

Part Number	Category	Size (22-31)	Left/Right
NFS3	5	26	L

Sandal Toe Ordering Example - NFS3526SL

Part Number	Category	Size	Sandal Toe	Left/Right
NFS3	5	26	S	L

Includes foot shell and Spectra® Sock

Category Selection - Highlander

Weight (kg)	Impact Level		
	Low	Mod	High
44-52	1	1	2
53-59	1	2	3
60-68	2	3	4
69-77	3	4	5
78-88	4	5	6
89-100	5	6	7
101-116	6	7	8
117-130	7	8	9
131-147	8	9	-
148-166	9	-	-

Product Specification

Weight Limit	Impact Level	Weight of Unit	Sizes (cm)	Build Height (cm)	Warranty
166kg		470g Size 25cm	22-31	10.8 (size 25)	3 years Foot shell - 6 months

Feet - PROTEOR - Freedom Highlander

Spare Parts Ordering Information

Part Number	Foot Shell Type
NFTC-3M-0 ▲●4-R ■	Standard, with CAP
NFTC-2M-1 ▲●4-S ■	Sandal Toe, without CAP



Cosmetic Attachment Plate

NCAP-3M-PO-▲●4-■




Spare Parts Ordering Guide

- ▲ = Size in cm (22-31 = Regular, 22-28 = Sandal Toe)
- = Skin Tone (L = Light, M = Medium, D = Dark)
- = "L" or "R" for Left or Right

Spectra® Sock	Colour	Size (cm)	Sock Size (cm)
NSO-NPS-20030-00	B	22-25	30
NSO-NPS-20035-00	B	26-28	35
NSO-NPS-20040-00	B	29-31	40

Colours: B = Black

Bumper Kit	Size (cm)
NKIT-00-11303-00	22-25
NKIT-00-11304-00	26-28
NKIT-00-11305-00	29-31

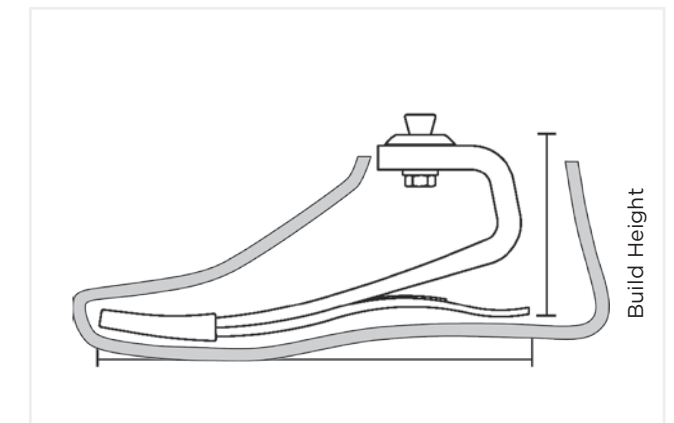

Technical Information

Build Height and Weight		
Size (cm)	Build Height (cm)	Foot inc. Shell
22-25	10.8	470g
26-28	12.1	590g
29-31	30	750g