



## SIZING OPTIONS FOR HEEL WEDGE

(Shown at actual size)

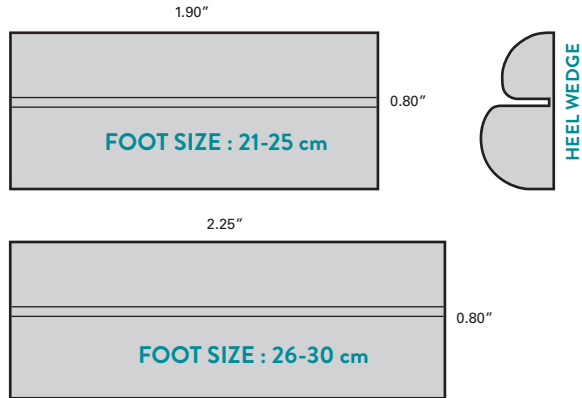

# INSTRUCTION SHEET

## ODYSSEY®<sub>K3</sub> WEDGE KIT

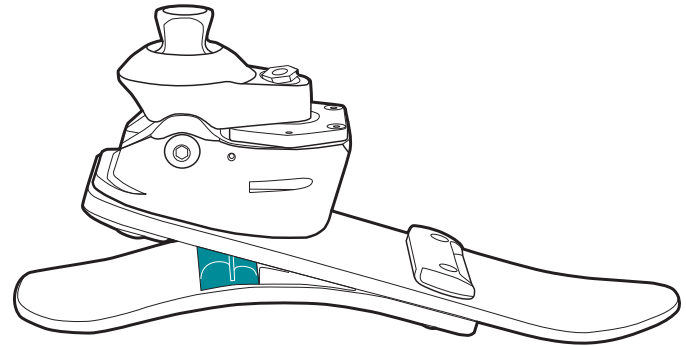

 college park  
800.728.7950 | 586.294.7950 | college-park.com


MADE IN THE USA  
521 INS OK3 WK 151222 

 COLLEGE PARK INDUSTRIES  
27955 College Park Industries, Warren, MI 48088 USA



EC REP EMERGO EUROPE  
Molenstraat 15, 2513 BH, The Hague, Netherlands


Heel wedges are not required.  
Follow these guidelines if an increase in firmness is desired.
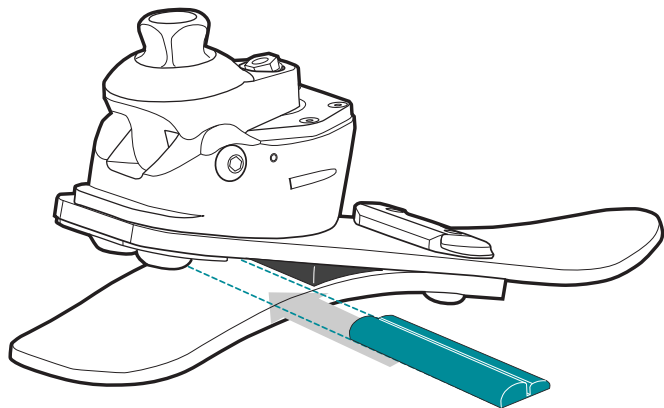
## ODYSSEY<sub>K3</sub> WEDGE PLACEMENT



## ADJUSTMENT OPTIONS FOR WEDGES

HEEL FIRMNESS	WEDGE
MEDIUM	NONE*
FIRM	 Single
X-FIRM	 Double

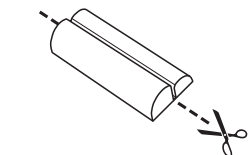

Make a Single Wedge by cutting the Double Wedge in half.

\*NOTE: Medium build comes standard. There is a triangular shaped wedge permanently bonded into all feet.

## WEDGE INSTALLATION AND REMOVAL

### SEMI-PERMANENT BOND (INITIAL FITTING)

- 1 Apply only the smallest amount of CA adhesive\* (included) necessary to temporarily hold the wedge into position. Too much will inhibit removal.
- 2 Install the rounded wedge directly behind the triangular wedge.
- 3 Press on the wedge and hold it for 15 seconds.



### WEDGE REMOVAL

To remove the rounded wedge without damage, pry it away with a finger or use a flat, blunt object such as a tongue depressor.

### PERMANENT BOND (FINAL FITTING)

- 1 Apply a thin line of CA adhesive\* around the bottom perimeter of the wedge.
- 2 When installing the rounded wedge, place directly behind the triangular wedge.
- 3 Press on the wedge and hold it for 15 seconds.

**NOTE:** The wedge will NOT be removable once it has been bonded.

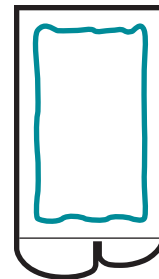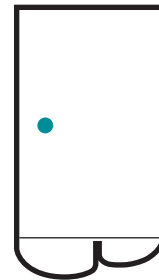

### CAUTIONS

- Use only the correct size wedge in the foot (see back for sizing guide).
- Do not remove permanent triangular wedge.
- Place wedge only in the recommended position.
- Do not substitute other objects in place of recommended wedges.

\*Cyanoacrylate (CA) adhesive, included as part of the wedge kit.