

Steeper Silicone Liner Overview

Transtibial & Transfemoral



Össur Transtibial Liners		First	First 3	Soft C	Sensitive 3C	Sensitive 6C	Protect 3C	Protect 3C Cushion	Relax 3C /3C Cushion	Relax 6C /6C Cushion
Indications		Primary prosthesis	Primary prosthesis	Good residual limb	Sensitive residual limb tenderness to pressure	Sensitive residual limb tenderness to pressure	Diabetic, dry, atrophic skin allergies	Diabetic, dry, atrophic skin allergies	Phantom pain, sensations, idiopathic residual limb pain	Phantom pain, sensations, idiopathic residual limb pain
Mobility Class 1-4		1, 2	1, 2	1, 2, 3, 4	1, 2	1, 2	1, 2, 3	1, 2, 3, 4	1, 2, 3, 4	1, 2, 3, 4
	Even residual limb shape, decreasing slightly distally, good soft-tissue cover	••	•	••	•	•	••	••	••	••
	Residual limb more bony, rather poor soft-tissue cover and/or with scars	•	••	•	•	••	••	••	•	••
	Residual limb more conical, good soft-tissue	•	••	•	••	•	••	••	••	•

Össur Transfemoral Liners		AKOS TFS	AKOS TFC	Relax TFS	Relax TFC	4Seal Classic TFS	4Seal Classic TFC	4Seal Senso TFS	4Seal Senso TFC	Key
Indications		Promotes grip between prosthesis and residual limb	Promotes grip between prosthesis and residual limb	Phantom pain, sensations, idiopathic residual limb	Phantom pain, sensations, idiopathic residual limb	Suction seal suspension plus phantom pain relief	Suction seal suspension plus phantom pain relief	Suction seal suspension plus phantom pain relief	Suction seal suspension plus phantom pain relief	<ul style="list-style-type: none"> • Suitable •• Particularly suitable
Mobility Class 1-4		1, 2, 3, 4	1, 2, 3, 4	1, 2, 3, 4	1, 2, 3, 4	1, 2, 3, 4	1, 2, 3, 4	1, 2, 3, 4	1, 2, 3, 4	
	TFS, even residual limb shape	••	•	••	•	••	•	••	•	
	TFC, conical residual limb shape	•	••	•	•	•	•	•	••	

Steeper Group, Unit 3 Stourton Link, Intermezzo Drive, Leeds, LS10 1DF
 Tel: +44 (0) 113 270 4841 E-mail: customerservices@steepergroup.com

www.steepergroup.com

STPPR055