


- Versatile finishing capability
- Sandal toe design allows shoe of choice
- Reduces fatigue and protects the residual limb
- Adjustable Stride Control™ to customize children's gait
- One year warranty

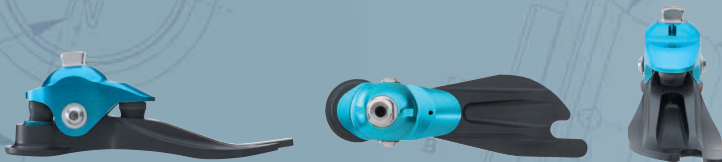

The College Park TruPer® Foot gives multi-axial ability and dynamic response in a kid-size package. Solving practical needs is where TruPer shines; it is simple to mount, is customizable for each individual, and easily accommodates the ever-changing child.




TruPer, Grown Up Function For Kids!™




RSLSTEEPER
Products Division
Unit 7, Hunslet Trading Estate
Severn Road, Leeds LS10 1BL
Tel: +44(0) 870 240 4133
Fax: +44(0) 870 240 4779
Email: sales@rslsteeper.co.uk
Web: www.rslsteeper.com

TruPer Order Information

TruPer Endo IP - NTP ■ ● EN▲

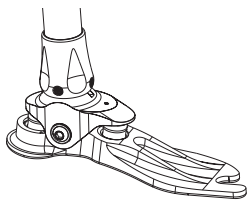


Sealing Boot	Size
NTP SB ■ ● 1618	16-18
NTP SB ■ ● 1921	19-21

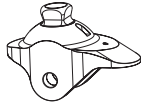

Endo Ankle	Size
NTP ENA 1618	16-18
NTP ENA 1921	19-21

■ = "L" or "R" for Left or Right
 ● = "C" or "N" for Caucasian or Negroid
 ▲ = Foot size in cm (16-21)
 (TruPer includes Shell, Sock and Sealing Boot)

TruPer Exo ALX - NTP ■ ● EX▲

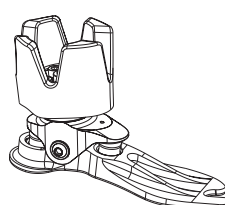


Pyramid Tool Kit	Size
NTP EXPK 22	16-21




Exo Block Kit	Size
NTP XBK ■ 1618	16-18
NTP XBK ■ 1921	19-21

Includes Mounting Bolt

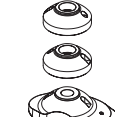


Growth Plate Kit	Size
NTP GK ■ ● 1618	16-18
NTP GK ■ ● 1921	19-21

Includes 2 Growth Plates, Mounting Bolt and Finishing Ring

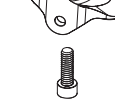

Exo Ankle	Size
NTP EXA 1618	16-18
NTP EXA 1921	19-21

TruPer Specifications

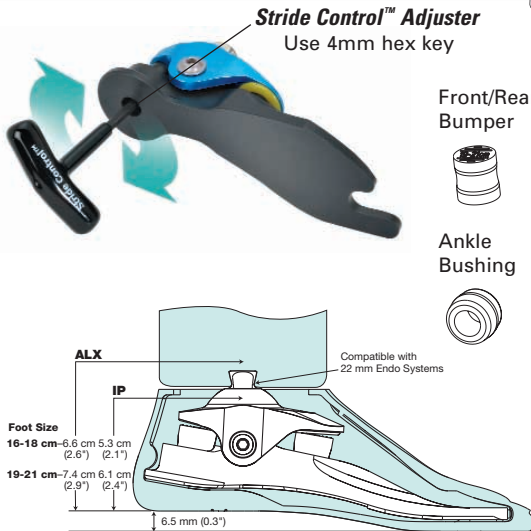

Adjustable Stride Control gives you the ability to customize an individual's gait timing, even with the Foot Shell on. The Stride Control Adjuster is used to fine-tune the foot's response by turning it clockwise/counterclockwise. This adjustment effects both plantar and dorsiflexion resistance without the need to change soft components.

TruPer Gait Matching Guidelines


	Weight		Front/Rear Bumper
	kg	lbs	
16-18cm	28-45	62-100	Firm
	21-27	46-60	Medium
	<21	<46	Soft
19-21cm	37-60	82-132	Firm
	23-36	51-79	Medium
	<23	<51	Soft

Growth Ranges

Weight Limit

45kg	60kg
18cm	21cm
17cm	20cm
16cm	19cm

Replace Foot Shell and Foreheel to increase size within growth range

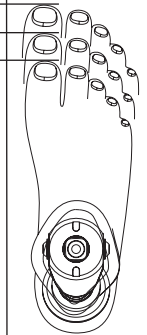

TruPer Features

Function

TruPer's inversion/eversion and superior transverse rotation provides exceptional ground conformance, improving performance and stability while walking or running.



Growth

The TruPer offers two growth ranges, 16-18cm and 19-21cm, and additional exoskeletal growth plates adding 2cm in height.




Cosmetics

TruPer's patented Sealing Boot interlocks with the Foot Shell, keeping debris away from the foot.



Sandal Toe

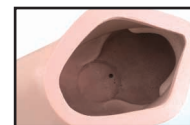

Heel Lock



Sealing Boot

