

- Multi-axial with rotation to easily adapt to any terrain
- Energy storing to reduce fatigue
- Adjustable Stride ControlTM to customize individuals' gait
- Smooth transition from high to low activities
- Exo and endoskeletal finishing capabilities
- Worry-free three year warranty



The College Park VentureTM Foot transitions with you...from the gym to the office or the trail to the pavement. The solution is finally here for the demanding lifestyle. The Venture is a paragon of precise engineering that generates performance to suit your dynamic lifestyle - without sacrificing comfort.



Venture, The Ultimate Fusion of Power and Agility.




RSLSTEEPER

Products Division
Unit 7, Hunslet Trading Estate
Severn Road, Leeds LS10 1BL
Tel: +44(0) 870 240 4133
Fax: +44(0) 870 240 4779
Email: sales@rslsteeper.co.uk
Web: www.rslsteeper.com

Venture Order Information

Venture Endo IP - NV ■ ● EN▲

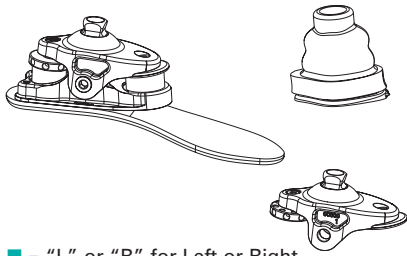


Sealing Boot	Size
NV SB ■ ● 2124	21-24
NV SB ■ ● 2526	25-26
NV SB ■ ● 2730	27-30

Endo Ankle	Size
NV ENA 2124	21-24
NV ENA 2530	25-30

■ = "L" or "R" for Left or Right
 ● = "C" or "N" for Caucasian or Negroid
 ▲ = Foot size in cm (21-30)
 (Venture includes Shell, Sock and Sealing Boot)

Venture Exo ALX - NV ■ ● EX▲

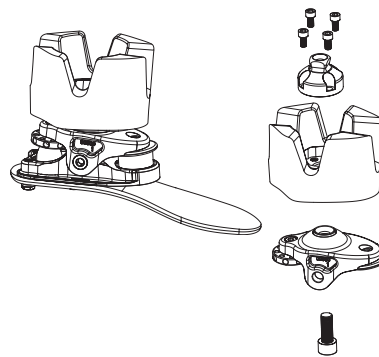


Pyramid Tool Kit	Size
NTS EXPK 30	21-30

Exo Block Kit	Size
NV XBK 2124	21-24
NV XBK 2526	25-26
NV XBK 2730	27-30

Includes Mounting Bolt

Exo Ankle	Size
NV EXA 2124	21-24
NV EXA 2530	25-30

Venture Specifications

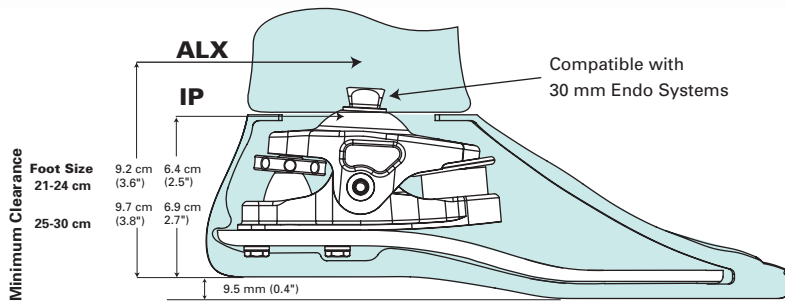

Venture Gait Matching Guidelines

Weight		Front Bumper	Rear Bumper
kg	lbs		
81-100	171-220	Firm	Firm
59-79	130-170	Medium	Medium
<59	<130	Soft	Soft


Bumper Size Ranges			
27-30 cm	A	B	
25-26 cm	A	B	
21-24 cm	B	C	

A, B, and C size range corresponds to Venture Foot sizes.

Adjustable Stride Control™




Adjustable Stride Control gives you the ability to customize an individual's gait timing, even with the Foot Shell on. The Stride Control Adjuster is used to fine-tune the foot's response by turning it clockwise/ counterclockwise. This adjustment effects both plantar and dorsiflexion resistance without the need to change soft components.

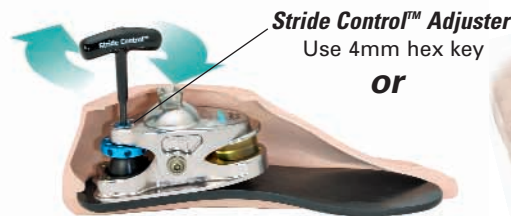

Venture Features

Function

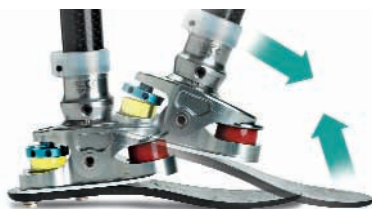

Energy Storing



Transverse Rotation
 Inversion-Eversion

Cosmetics



Venture's patented Sealing Boot interlocks with the Foot Shell, keeping debris away from the Foot.



Venture's Foot Shell is anatomically designed with Spectra® reinforcement.

